Petite&Mars

GAIA

GAIA Art. code 325035

GAIA DUO Art. code 327035

HANDS-FREE WEARABLE ELECTRIC BREAST PUMP

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ON / OFF

Long Press to Turn ON/OFF, Short Press to Pause/Play


**Vacuum
down**

**Vacuum
up**

Mode Switch

Stimulation / Expression / 2 in 1

+ - / 3 sec = LOCK / UNLOCK BUTTONS

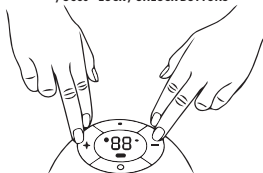C ZAČÍNÁME / ZAČÍNÁME / ELSŐ LÉPÉSEK / PIERWSZE URUCHOMIENIE / GETTING STARTED / ERSTE SCHRITTE / ÎNCEPEREA UTILIZĂRII / MISE EN ROUTE / PRIMA CONFIGURAZIONE

Do not get wet!

C1

C2

Washable
components

C3

Type-C
Cable

C4

25mm

C5

15mm

17mm

19mm

21mm

C6

03

02

02


C7


C8

C9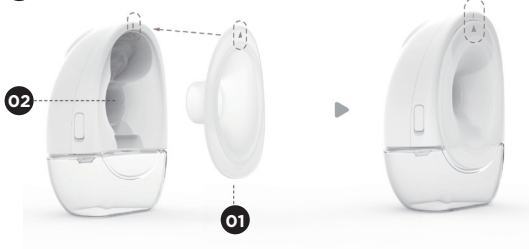


C10

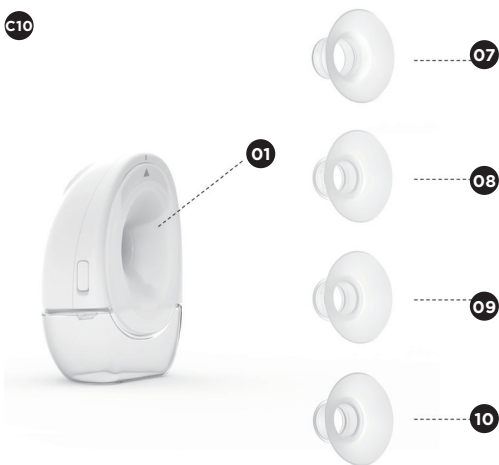
D POUŽITÍ PŘENOSNÉ ODSAVAČKY / POUŽITIE PRENOSNEJ ODSÁVAČKY / HORDOZHATÓ MELLSZÍVÓ HASZNÁLATA / UŻYWANIE LAKTATORA / USING WEARABLE PUMP / VERWENDUNG DER WEARABLE-PUMPE
UTILIZAREA POMPEI WEARABLE / UTILISATION DU TIRE-LAIT WEARABLE / UTILIZZO DEL TIRALATTE WEARABLE

D1

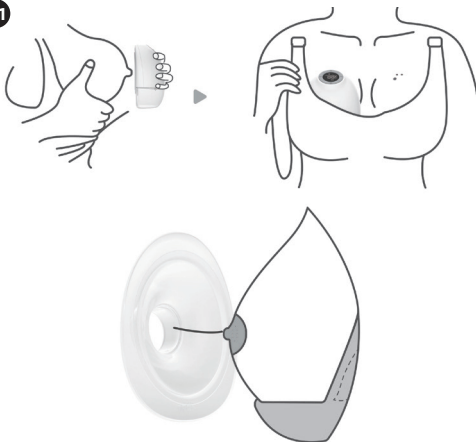D2

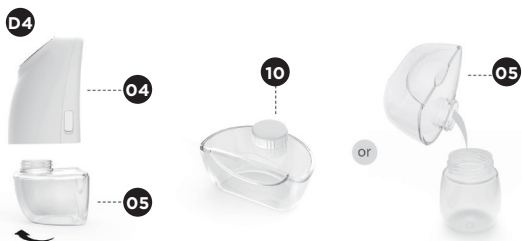D5

D6

Wearable, silent, hands-free, comfortable

Thank you for choosing our Petite&Mars GAIA breast pump. We wish you a satisfying and trouble-free experience with the device and complete satisfaction with its performance.

1. PRODUCT DESCRIPTION

1.1 Parts – A

- | | |
|---------------------------|---------------------------|
| 01. 25 mm Silicone Flange | 09. 17 mm Insert |
| 02. Breast Shield | 10. 15 mm Insert |
| 03. Membrane | 11. Bottle Lid |
| 04. Pump Motor | 12. Converter |
| 05. 180 ml Bottle | 13. Type-C Charging Cable |
| 06. Valve | 14. Bottle Brush |
| 07. 21 mm Insert | 15. Bra Adjuster |
| 08. 19 mm Insert | |

1.2 Buttons control - B

1. ON/OFF – Long press to turn the pump ON/OFF, short press to pause/resume.
2. Mode Switch – Stimulation / Expression / 2-in-1
3. Vacuum Down
4. Vacuum Up

Note: This product has a built-in lithium-ion battery. When the battery indicator shows only one bar, we recommend using a certified 5 V 1 A power adapter to fully charge the pump motor. When the battery indicator flashes during operation, it indicates that the pump motor should be charged as soon as possible.

2. INTENDED USE

2.1 Indications for use

Powered breast pump designed to express and collect breast milk from lactating women, to help alleviate breast engorgement, maintain milk supply, and provide breast milk for future feedings when separation of mother and baby occurs. The device is intended for single-user use only.

2.2 Operating life

The operating life of the device is not less than 500 hours.

The expected operating life of the replaceable washable components is approximately three to six months, depending on how frequently the parts are used and washed. It is recommended to replace the washable components after long-term storage or before using the pump for your next child.

3. SAFETY

3.1 Warnings

1. This is a single-user product; use by more than one person may present a health risk and void the warranty.
2. It is recommended to consult with a licensed healthcare professional before using this wearable pump if you have an active implanted device that could be affected by a magnetic field (e.g. pacemaker, implantable cardioverter-defibrillator).
3. Inspect for damage, and clean and sterilize all washable components that come into contact with your breast and breast milk prior to first use.
4. Clean all parts that come into contact with your breast and breast milk and inspect them for damage after every use.

5. During normal use, areas of the wearable pump may warm up. If your pumping environment is at the upper operating temperature limit (+30°C / +86°F) for the maximum session time (30 minutes), the temperature of the may reach up to +45°C / +113°F.
6. Do not store wet or damp parts, as mold may develop.
7. Do not immerse the pump motor in water or other liquids. Wipe with a clean, damp cloth.
8. This product contains small parts and a cable that may present a hazard. Keep out of reach of children.
9. Only use manufacturer-supplied parts with this wearable pump, including the charging cable. Do not substitute with non-approved accessories.
10. It is recommended to use an adapter compatible with both IEC 60601-1 and your regional power source.

3.2 Precautions

1. Always inspect all parts for damage and cleanliness before use.
2. Always ensure all parts are completely dry before assembling.
3. Do not place the pump motor in the fridge or freezer.
4. This wearable pump and its components are not heat-resistant. Keep them away from heat sources.
5. Do not use flammable materials while using this wearable pump or while it is connected to a charger.
6. Do not use this wearable pump while sleeping or if you are overly drowsy.
7. Using a breast pump should not cause pain. Do not try to express with an intensity setting that is too high or uncomfortable. If excessive discomfort or pain is felt during pumping, stop pumping, break the seal of the breast shield around your nipple and discontinue the session.
8. Contact your healthcare professional if you express minimal or no milk, or if expression is painful.
9. For assistance with correct breast shield sizing and comfort, please see section 4.3 or talk to a lactation consultant or breastfeeding specialist.
10. Extra care is necessary when using this wearable pump in the vicinity of children.
11. This wearable pump should not be used while lying down or engaging in strenuous exercise or activities with risk of harm.
12. Always break the seal of the cushion around your nipple before removing this wearable pump from your breast.
13. Do not expose this wearable pump or its components to direct sunlight.

3.3 Contraindications

1. Do not use the breast pump if you have an active breast infection, mastitis, breast abscess, cracked or bleeding nipples, or unexplained breast pain.
2. Do not use the pump on damaged, irritated or broken skin.
3. If you have recently undergone breast surgery, consult your healthcare professional before use.
4. This device should not be used if you experience dizziness, fainting sensations, or circulatory problems during pumping.

3.4 Battery safety warnings

1. Do not expose the pump motor or battery to direct sunlight, fire, or temperatures above +45°C / +113°F.
2. Do not attempt to open, puncture, crush or modify the battery.
3. If the pump motor becomes unusually hot, emits an odor, or shows signs of swelling, stop using it immediately.
4. Only use certified 5V 1A adapters. Using third-party fast chargers (e.g. 9V/12V) may damage the battery and void the warranty.
5. Do not charge the device near water, sinks, or other wet environments

4. GETTING STARTED

4.1 Cleaning and sterilizing

For the first use and after each use, you must disassemble, wash and sterilize all breast pump parts that come into contact with breast milk.

Disassemble and clean the parts that come into contact with breasts and breast milk immediately after use to avoid residual breast milk that may cause bacteria to grow.

Take care when cleaning the silicone cushion, diaphragm and valve. If any of these parts are damaged, your breast pump will not function properly.

Never wash or sterilize the pump motor. This component does not come into contact with your breast milk. You can wipe this part clean with a soft cloth. Do not allow it to get wet. **C1**

The pump motor contains electronics that should be kept dry. A clean, damp cloth can be used to clean the pump motor. Any other cleaning method will be considered misuse under the warranty.

Washable components C2

- | | |
|---------------------------|------------------|
| 01. 25 mm Silicone Flange | 07. Converter |
| 02. Membrane | 08. 15 mm Insert |
| 03. Breast Shield | 09. 17 mm Insert |
| 04. Valve | 10. 19 mm Insert |
| 05. 180 ml Bottle | 11. 21 mm Insert |
| 06. Bottle Lid | |

Clean all washable components that come into contact with milk to prevent the build-up of dried milk residue, which may cause the growth of bacteria and may be harmful to health.

Warnings: Please take care when washing and sterilizing all parts.

Clean washable components:

- Rinse in cold, clear water (approx. +20°C / +68°F).
- Wash in warm water (approx. +30°C / +86°F) using mild dishwashing soap suitable for baby items and wipe afterwards with a clean cloth.
- Rinse again with cold, clear water (approx. +20°C / +68°F) for 10 to 15 seconds.
- Alternatively, place the washable components on the top shelf of the dishwasher (if suitable).

Sterilize washable components:

- Boil in water for no more than five minutes,
- or**
- Sanitize by using boiling water or a steam sterilizer bag (sold separately),
- or**
- Clean the pump kit parts with room temperature water and baby bottle sterilizing tablets, according to the tablet manufacturer's instructions.

Material information

- The bottle is made from Tritan™, a BPA-free material. Do not sterilize Tritan bottles in steam sterilizers above 110°C / 230°F, as exposure to high heat may reduce durability.
- All silicone parts are heat-resistant but may deform if pressed, stretched or twisted during sterilization.
- Replace silicone parts immediately if they show clouding, whitening, tears, or reduced elasticity.

Replacement of parts

Regularly inspect all washable components. Replace parts every 3–6 months or sooner if you notice reduced suction, visible damage, odor retention, or milk leakage.

Use only original replacement parts to ensure proper suction and hygiene.

4.2 Charging

When the battery is low during operation, the battery indicator light will flash.

When charging, the indicator will light up and the pump will not turn on. Use the supplied charging cable to connect the Pump Motor to a USB power source (5V 1A), such as your phone adapter. When fully charged, all battery indicator bars will light up. It takes approximately 2 hours to fully charge the battery, which provides approximately 2.5 hours of usage time at standard settings.

Warning: Please only use the charging cable provided with the wearable pump. Do not use the wearable pump while it is charging. **C3**

Tip: Charge immediately after finishing your pumping session if a low battery is indicated.

4.3. Breast cushion / inserts sizing

This wearable pump comes with a pre-assembled 25 mm silicone cushion and four additional silicone inserts in sizes 21 mm, 19 mm, 17 mm and 15 mm, so you can find a fit that is comfortable for you.

Standard cushion: 25 mm **C4**

Available accessories included in the box: C5

- 15 mm
- 17 mm
- 19 mm
- 21 mm

4.4. Assembling the pump

1. Make sure all parts are dry before assembling them (after the parts have been cleaned and sterilized).
2. Attach the diaphragm (03) to the breast shield (02). Attach the valve (06) to the breast shield (02). **C6**
3. Push the fully assembled breast shield (A) into the pump motor (04). Make sure it is assembled in place and the diaphragm is pressed tightly. **C7**
4. Twist the bottle (05) onto the pump motor (04). Pay attention to the screw-in position. **C8**
5. Push the cushion (01) into the breast shield (02). Make sure it is aligned and assembled securely in place. **C9**
6. If the standard 25 mm silicone cushion (01) is too big for you, you may choose the 21 mm (07), 19 mm (08), 17 mm (09) or 15 mm (10) silicone insert and place it in the center of the 25 mm cushion (01). **C10**

5. USING WEARABLE PUMP

5.1 Aligning your nipple

Correct nipple alignment in the cushion is key to comfort, efficient milk expression and performance. Your nipple must be positioned correctly in the cushion tunnel before pumping to avoid discomfort. **D1**

5.2 Finding the right compression

This Wearable Pump is designed to be worn with your standard nursing bra and can be used in either a sitting or standing position.

Getting the right compression may impact milk volume. This wearable pump needs to be firmly pressed against your breast; there should be no gaps between the wearable pump and your breast.

Every bra has a different level of support, so make sure to adjust the compression before each session. If you are using a particularly structured and inflexible bra, the included bra adjuster may be used to create more room.

Once the buckle at each end is attached to the nursing bra, adjust the length of the bra adjuster to achieve a comfortable fit. **D2**

5.3 Starting your session D3

- a. Stimulation Mode
- b. Expression Mode

- c. Battery Bar
- d. Level Indicator and Timer
- e. 2 in 1 Mode

“ON/OFF” button

Long press the “ON/OFF” button to switch the pump on. The LED screen will light up and the pump will start to work.

During operation, short press the “ON/OFF” button to pause or resume if you need to pause pumping for any reason. When the pause time exceeds 10 minutes, the pump will automatically shut off.

During operation, the LED screen remains illuminated, allowing you to monitor the device status at a glance.

M Mode

After you long press the “ON/OFF” button, the pump will automatically start in Stimulation Mode at the level used in the last pumping session.

For the first use of the pump, the default setting is Level 1 in Stimulation Mode.

After two minutes in Stimulation Mode, the device automatically switches to Expression Mode at preset suction Level 3.

You can also manually press the “M” button to switch between “Stimulation”, “Expression” and “2-in-1” mode.

 Stimulation Mode

 Expression Mode

 2-in-1 Mode (5 quick suction cycles + 5 deep and slow suction cycles).

Vacuum Settings

The pump has 9 vacuum settings, giving you control over what feels comfortable and works most efficiently in Stimulation, Expression and 2-in-1 Mode.

Once you feel suction on your nipple, use “+” or “-” to adjust the vacuum level and find your preferred setting in all modes.

The screen always displays the timer. To check the current suction level, briefly press the “+” or “-” button; the level will appear for 5 seconds before the display returns to the timer.

Memory function: The pump remembers the vacuum level of each mode from the last time used, making it easier for you to start your next pumping session.

Ending your session

Long press the “ON/OFF” button to stop pumping and switch the device off. (The pump will automatically switch off after 30 minutes of continuous operation).

Unclip your bra and use your finger to break the seal of the cushion around your nipple to avoid nipple damage.

Remove the wearable pump from your bra.

5.4 Storing your milk

After finishing your pumping session, twist the Bottle (05) to remove it from the pump motor (04). You may store your expressed milk in the bottle using the lid (11) provided or pour your expressed milk into a container of your choice for storage or feeding. D4

When storing milk, always follow the guidance of your lactation consultant or healthcare professional.

Warnings:

- Clean all parts that come into contact with your breast and breast milk and inspect them

for damage after every use.

- Do not store wet or damp components, as mold may develop.

Recommended breast milk storage guidelines

Always follow the advice of your healthcare professional. If no specific guidance is provided, you may use the following general

Recommended breast milk storage guidelines

Always follow the advice of your healthcare professional. If no specific guidance is provided, you may use the following general recommendations:

Storage Location	Temperature	Maximum Duration
Room temperature	up to 25°C / 77°F	4–6 hours
Refrigerator	0–4°C / 32–39°F	up to 4 days
Freezer (separate door)	–18°C / 0°F	6–12 months

Thawed breast milk must not be refrozen.

Gently swirl (do not shake vigorously) before feeding to mix any natural separation.

5.5 Using converter and nipple set to feed baby

Twist the converter onto the bottle and then twist the nipple set onto the converter. This creates a feeding bottle set. **D5**

Please note that the nipple set is NOT included with this product.

Please use a nipple set that fits wide-neck baby bottles.

5.6 How to disassemble?

Gently pull the assembled set forward and away from the pump motor to remove it. Disassemble all washable components for cleaning and sterilization. **D6**

Replacement of parts

Regularly inspect all washable components. Replace parts every 3–6 months, or sooner if you notice reduced suction, visible damage, odor retention or milk leakage.

Use only original replacement parts to ensure proper suction and hygiene.

6. TECHNICAL SPECIFICATIONS

Power: 5 V – 1 A

Battery type and capacity: Li-ion Polymer Battery

Specification: 3.7 V 1300 mAh

Stimulation speed: 100 CPM (Cycles Per Minute) (+/- 5 CPM)

Expression speed: 35–90 CPM (Cycles Per Minute) (+/- 5 CPM)

Stimulation vacuum: –0.004 – –0.016 MPa, 9 levels adjustable (+/- 0.003 MPa)

Expression vacuum: –0.004 – –0.032 MPa, 9 levels adjustable (+/- 0.003 MPa)

Timer: 30 minutes

Noise level: ≤ 55 dB(A)

Operating conditions: Temperature 5°C–40°C, humidity 10%–90% RH

7. TROUBLE SHOOTING

Problem 1: Weak suction or no suction

Cause: Check whether the is installed incorrectly or damaged.

Solution: Reinstall or replace the .

Cause: Check whether the connection between the and Pump Motor is tight.

Solution: Tighten all connections.

Cause: Check whether the fits the breast completely.

Solution: Ensure the fits the breast at all times with the nipple centered in the Breast Shield tunnel.

Problem 2: Cannot turn on or charge

Cause: Check whether the output of the charger meets the requirements and whether the charger is plugged in properly.

Solution: Use the correct charger (5 V 1 A) and make sure it is properly connected.

Cause: The lithium battery is completely discharged.

Solution: Charge the pump when the low battery indicator appears.

Problem 3: Milk backflow

Cause: The is not installed properly or is damaged.

Solution: Immediately stop pumping and switch off the pump, then clean it. Do not use the pump again until the has been correctly reinstalled or replaced with a new one.

DISPOSAL AND RECYCLING (WEEE COMPLIANCE)

This product contains electronic components and a rechargeable lithium battery. Do not dispose of the device with household waste. Dispose of the Pump Motor and battery at designated electronic recycling facilities, in accordance with local regulations.

Packaging materials and plastic components may be recycled where facilities exist.

SYMBOLS

	"Do not dispose of with household waste" (WEEE symbol)
	"Refer to instruction manual"
	"Temperature limits"
	"CE" – Conformity marking

WARRANTY CONDITIONS

1. The warranty period is 24 months from the date of purchase.
2. The warranty covers manufacturing defects detected during the warranty period.
3. When submitting a complaint, attach the proof of purchase and a properly completed warranty card.
4. Use the product only for its intended purpose.
5. Store and maintain the product properly, according to the instructions in this manual.
6. Submit warranty claims only through the store where you purchased the product.
7. Have the product repaired only by an authorized service center.
8. Deliver the product for repair clean and properly packed.

THE WARRANTY DOES NOT COVER

1. Defects caused by failure to comply with warranty or operating conditions.
2. Damage resulting from improper use or maintenance.
3. Parts subject to normal wear and tear.
4. Mechanical damage caused by dropping or impact.
5. Plastic or fabric components that faded or discolored due to UV exposure or cleaning.

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